

Shopping Basket



The Shopping Basket is designed for storage of general items not medical equipment.

The 600D PVC polyester fabric is rip and tear resistant however **DO NOT** store items with sharp edges or fold the pushchair without first removing any items, failure to do this may damage the Shopping Basket.

NOTE. If your pushchair is difficult to fold or unfold it may be caused by the Shopping Basket being tangled in the frame. To resolve this remove the Basket DO NOT force the frame.

WARNING: Maximum load 2kg (4.4lbs). Overloading may make the pushchair unstable.

Fitting

The Shopping Basket is fitted below the seat base, resting on the bottom 'X' frame and secured to the front legs and seat support tubes by popper straps.

- Unfold the Shopping Basket and place it on the 'X' frame under the seat with the lowest edge of the basket at the front.
- Wrap the front corner straps around each front leg just above the highest footrest pin and secure the poppers.
- Wrap the rear corner straps around each seat support tube above the rear leg brackets and secure the poppers.
- Wrap the rear centre straps around the centre cross bars and secure the poppers.

Folding or unfolding the pushchair may cause a fixing popper to come undone.

Maintenance

The 600D PVC polyester fabric is stain resistant, it can be cleaned under a warm running tap or wiped clean. DO NOT use bleach or machine wash.