

## Replacement Spacer Bar (front folding frame)

The spacer bar joining the upper and lower folding frame behind the footplate is attached at the top and bottom with rivets. These rivets are replaced by user friendly bolts for the replacement spacer bar.

### Contents

- 1 x Replacement Spacer Bar
  - 1 x Bolt 25mm x 5mm inc Washer & Nyloc Nut
  - 1 x Bolt 20mm x 5mm inc Washer & Nyloc Nut
  - 1 x Allen Key
- (+ 2 x spare bolts just in case)

### Fitting Tips

#### 1. Preparation

It can be useful to take a photo of how the pieces fit together so you can refer to this when fitting the replacement spacer bar.

#### 2. Removal

You will need to remove the rivets holding the existing bar. It will not be possible to drill them off (they simply turn) you will need to either cut off the rivet head or file it off. An oscillating multitool works well.

Once the rivet head is removed push the other end of the rivet out. A nail and hammer can help with this.

#### 3. Fitting

Position the replacement spacer bar and fit nuts and bolts provided using the allen key and a spanner. The joint needs to be firm enough that it is not loose but the folding frame still able to move freely.

Note. The nuts are 'nyloc' they have a nylon lining so they don't need to be done up tightly to resist vibration.